

## Personal Pendant

The PalatiumCare single-button, water-resistant personal pendant is an ideal choice for residents in all levels of care. The light-weight and easy-to-use design allows residents to go about their days without being hampered by bulky devices. When an emergency takes place, no matter where they are, residents can call for help.

### Features:

- Can be worn in the shower or bathtub
- Breakaway neck cord
- Fully supervised for reliability
- LED visual indicator
- Monitored for low battery condition
- Replaceable battery
- Standard & Advanced Locating available

By using a device with a replaceable battery, your community can extend the life of a single device for years, saving hundreds of dollars on each resident over the life of your nurse call system.

The pendant can also be worn around the wrist using adjustable, velcro wrist straps. For more information, contact PalatiumCare or your local dealer.



### Specifications

- Dimensions: 2 1/2" x 1 7/8" x 3/4"
- Color: Beige
- Power: Panasonic CR2450
- Weight: Less than 1.3oz. (40gm)
- Compatibility: Works with all PalatiumCare Wireless Nurse Call Systems

To replace the battery of your PalatiumCare Personal Pendant, remove the plastic necklace by unsnapping the clasp.



Remove three small screws. Be sure to place them somewhere secure so as not to lose them.



Remove the back cover from the pendant.



Replace the battery. For longest life, make sure the new battery has not touched another battery.

